

HOW TO DELETE PREVIOUS DATA

- Insert the device on the docking station and locate the device drive, EX: HPR11438098 (F:)
- Right click on device and select format
- Check off format
- Select OK on the Warning pop up window.

THE PATIENT REMOVED THE MONITOR EARLY

At your discretion, you can send in the data or have the patient repeat the test. Davis Medical will contact you if the data is insufficient and verify if you would want the data processed. For short recordings we recommend having the patient redo the study for an optimal quality report.



CLEANING

To clean the recorder:

- Wipe the exterior of the Trident 30 and its accessories with a damp cloth.
- Use mild detergent diluted in water or surface wipes such as PDI Super Sani-Cloth® Germicidal Disposable Wipes ("purple top").
- Take care not to allow water or moisture to enter the Trident 30, especially around the cable contacts.
- Other acceptable cleaning agents: up to 97% isopropyl alcohol, up to 3% benzyl ammonium chloride.
- Dry with a clean cloth or paper towel.